

Positive Patient Identification Procedure on ABL or AQT Devices

A patient with KNOWN Identification Number (MRN)

1. Enter /Scan patient MRN
2. Patient demographics is displayed automatically, refer to picture below

Patient ID	1060625
Patient last name	TESTPATIENT
Patient first name	Pathnet lis2
Date of birth	25/12/1967
Sex	Male

If correct - continue

Patient ID	1224215
Patient last name	[REDACTED]
Patient first name	[REDACTED]
Date of birth	[REDACTED]
Sex	Unknown

If incorrect- enter correct patient MRN at the time of testing. *Otherwise fill out the Unknown patient/incorrect patient details Correction form and send the form to the local lab or your Hub Co-ordinator. **Contact your PoCT or Hub Co-ordinator if an incorrect Patient ID has been entered at the Point of care testing. **site specific*

An UNKNOWN Identification Number (MRN)


1. Scan 'UNKNOWN MRN' barcode in Patient MRN

The UNKNOWN MRN barcodes are:

- device specific
- single test only.



2. Enter patients details (patients last name, first name, DOB in DDMMYYYY and gender)
3. Obtain an extra result printout
4. Stick the scanned barcode on the "**Unknown patient/incorrect patient details Correction form**". Fill out the form with all patient details

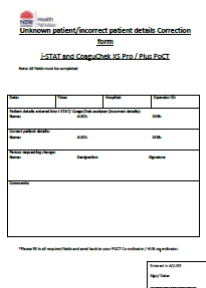

5. Attach the printout to the completed **Unknown patient/incorrect patient details Correction form**

6. Leave the forms in the designated PoCT tray or
7. Send the form to your PoCT or HUB Co-ordinator



Follow your site procedure