

**Sample Analysis on ABL90FLEX**

- 1. Scan or enter user logon barcode
- 2. Ensure the analyser in on 'Ready' mode
- 3.

LIFT



Lift inlet position to either syringe or capillary mode.

- 4. Introduce the sample

PUSH



HOLD



Hold syringe firmly against inlet and push up.

For capillary, hold sample horizontal and allow it to aspirate.

- 5.

REMOVE



Remove the syringe/capillary as prompted by the analyser.

- 6.

CLOSE INLET



Close the inlet

- 7. Enter correct patient MRN – patient demographics will be displayed
- 8. Select *Sample type/cord blood type*
- 9. Results are printed automatically once analysis is completed – check results and demographics are correct. *Follow non-conformance procedure if data is incorrect*
- 10. Log off analyser.